


Products:

3M Facepiece Respirator 6000 Series				
	For Use by:	Reusable:	Cartridges:	After use:
	DTF members	Yes	Replace as needed	Must decon
	Hazmat Members	Yes	Replace as needed	Must decon

3M Facepiece Respirator 6000 Series: issued one facepiece and P100 cartridge

- The surgical mask and N-95 respirators are acceptable face coverings for the work performed by the Disinfecting Task Force (DTF), but the 3M Half mask with cartridge was issued to each team member for comfort and durability.
- If a member is issued this 3M Half Mask, they will be only tested for fit by the Hazmat Division per local OSHA Office and is **not approved for any other use** besides cleaning by Disinfecting Task Force.
- The hard-plastic case should be wiped down with a disinfectant wipe after each DTF station visit and allowed to air dry. Avoid getting the filter media wet.
- Replace the cartridge, if it becomes hard to breathe through, damaged, dirty, or wet.

How to clean a 3M Half Facepiece Respirator 6000 Series-

<https://multimedia.3m.com/mws/media/17939590/cleaning-and-disinfecting-3m-reusable-respirators-following-potential-exposure-to-coronaviruses.pdf>

3M Half Facepiece Respirator 6000 Series Training Videos-

<https://multimedia.3m.com/mws/media/18149330/training-videos3m-half-facepiece-reusable-respirator-6000-series.pdf>

3M Facepiece Respirator 6000 Series Product Info-

https://www.3m.com/3M/en_US/company-us/all-3m-products/~/3M-Half-Facepiece-Reusable-Respirator-6100-07024-AAD-Small-24-EA-Case/?N=5002385+8709322+8711405+3294780296&preselect=8720539+8720550+8720785&rt=rud